

Presenting our **NEW** Burnaid® offering –



Burnaid® Hand Dressing

- Part of the trusted Burnaid® family
- Designed with consultation of key industry participants
- For use on hands or lower arm burns
- Universal dressing – one size fits all
- Easy to apply
- Clear instructions for use, on packaging



Application Guide – for best results using the Burnaid® Hand Dressing

1

Choose product

- Burnaid® Hand Dressing
- Burnaid® Cling Wrap



2

Lay out Burnaid® Hand Dressing a on flat surface



3

Cut or tear open product (at labelled area)



4

Prepare Dressing for insertion of hand

- Keep dressing on a flat surface
- Hold opened product and beginning of top foam insert with one hand
- Place affected hand into dressing, between both foam inserts



5

Secure Dressing

- Using Burnaid® Cling Wrap



Burnaid®

Code

BDH1

Description

Burnaid® Hand Dressing
60cm x 22cm

Available in Packs of 10
or as single units

implox Pty. Ltd.
HEALTHCARE

Units 22-24
60-66 Richmond Road,
Keswick 5035, South Australia
Telephone: 8351 1455
Facsimile: 8293 7377
Email: CustomerService@implox.com
www.implox.com